

Exclusively distributed by



SAFE LIFTING SOLUTIONS

FOR POST FALL MANAGEMENT

PRODUCT EXPERTISE

THERAPEUTIC SOLUTIONS

SERVICE EXPERTISE



2024/2025

WIN  CARE





"Our team use a Mangar Lifting Cushion to raise the fallen person from the floor. The cushions provide a safe lift for the resident but also help prevent musculoskeletal injury to our staff. Because of the amount of falls we attend, the lifting cushions are used every day and have been for nearly 10 years. They are a reliable piece of equipment that we wouldn't be without."

Heather Rimmer, Norfolk Swift Response

Contents

- 4. The Mangar Eagle
- 6. The Mangar Camel
- 8. The Mangar Elk
- 10. The Mangar Rhino
- 12. Protecting Healthcare Professionals
- 13. Contact Us



MANGAR Eagle Lifting Cushion
LSS403800



MANGAR Camel Lifting Cushion
LSS403810



MANGAR ELK Lifting Cushion
LSS403820



MANGAR Rhino Lifting Cushion
LSS403830

Contact us today

For more information visit



1300 133 120 | aidacare.com.au
sales@aidacare.com.au



MANGAR Eagle Lifting Cushion

The Eagle lifting cushion will both sit up and lift a fallen person. It has a small footprint and can be used in small spaces such as a bathroom.

EASY TO FIT

- Lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for indoor and outdoor use
- Can be used independently or assisted

SIMPLE TO MAINTAIN

- Clean using a proprietary non-abrasive mild liquid cleaner
- Easy to deflate, fold away and store in the carry bag provided
- Annual load testing is not required
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

SAFE TO USE

- Lifts the frailest person up to 35 stone/ 220 kg from the floor
- Exceptionally rigid and supportive to back, neck and head
- Ideal for use in supported living apartments and home care

IMPROVES WELL-BEING

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the carer
- The backrest gives additional support to raise the person from lying to sitting
- Small footprint is ideal for use in a home environment

TECHNICAL SPECIFICATIONS

Max user weight	220 kg
Max seat height	56 cm
Min height	Flat
Seat depth	57 cm
Total length	112 cm
Width	64 cm
Total product weight	4.5 kg
Back/headrest length	52 cm
Airflo Compressor	24
Lifting performance per charge	up to 6 lifts



Features

- 1. Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the Eagle.
- 2. The backrest** provides support to the back and head.
- 3. Made from very strong polyurethane coated nylon material.**
- 4. Each layer inflates individually** ideal for single-handed care
- 5. Sides of the Eagle can be folded up** to allow manoeuvring into tight spaces.

Accessories

- Slidesheet
- Eagle Carry Bag



Product code: LSS403800



SCAN ME



MANGAR Camel Lifting Cushion

The Camel Lifting Cushion will both sit up and lift a fallen person.

EASY TO FIT

- Lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for indoor and outdoor use
- Can be used independently or assisted

SIMPLE TO MAINTAIN

- Clean using a proprietary non-abrasive mild liquid cleaner.
- Easy to deflate, fold away and store in the carry bag provided
- Annual load testing is not required
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

SAFE TO USE

- Lifts the frailest person up to 320 kg from the floor
- Stable and supportive
- Ideal for carers of individuals with neurological disorders who fall regularly

IMPROVES WELL-BEING

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the carer
- Individuals with a cognitive impairment are less distressed when lifted with a Camel
- The backrest gives additional support and is particularly suitable for bariatric patients

TECHNICAL SPECIFICATIONS

Max user weight	320 kg
Max height	56 cm
Min height	Flat
Seat depth	47 cm
Total length	112 cm
Width	70 cm
Total product weight	6.5 kg
Back/headrest length	81 cm
Airflo Compressor	24
Lifting performance per charge	up to 4 lifts



Features

- 1. Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the Camel.
- 2. The head should lie on the Camel badge** and the body should lie centrally.
- 3. The backrest** may be partially inflated before the lower sections.
- 4. Made from very strong polyurethane coated nylon material.**
- 5. Each layer inflates individually** ideal for single-handed care.

Accessories

- Slidesheet
- Camel Carry Bag



Product code: LSS403810



Scan me



MANGAR ELK Lifting Cushion

The ELK is a lifting cushion designed to lift from the floor.

EASY TO FIT

- Lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for use inside or out
- Can be used by one carer

SIMPLE TO MAINTAIN

- Clean using a proprietary non-abrasive mild liquid cleaner.
- Easy to deflate, fold away and store
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments
- Annual load testing is not required

SAFE TO USE

- Lifts the frailest person up to 450 kg from the floor
- Exceptionally rigid and stable
- Ideal for carers of individuals with neurological disorders who fall regularly

IMPROVES WELL-BEING

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the carer
- Can be used in confined spaces where traditional lifting equipment may not fit

TECHNICAL SPECIFICATIONS

Max user weight	450 kg
Max height	56 cm
Min height	Flat
Seat depth	50 cm
Width	57 cm
Total product weight	3.6 kg
Airflo Compressor	24
Lifting performance per charge	Up to 9 lifts



Features

1. **4-Way Hand Control.**
2. **Made from very strong polyurethane coated nylon material.**
3. **Remove or keep stability bar** to provide extra comfort for the user.
4. **Can be used to lift a stretcher.**
5. **Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the ELK.
6. **Powered by an Airflo 24 Compressor.**

Accessories

- ELK Stretcher Bars
- Slidesheet
- ELK End Cap Set



Product code: LSS403820



Scan me



MANGAR Swift Transfer Slide

Transfer and reposition patients in the supine position with minimal moving and handling force needed, while providing a stable, safe experience for the patient.

Maximum User Weight: 500 kg (1102 lbs/79 st)

Inflated Height (approx): 14 cm (5.5")

Overall Length (approx): 204 cm (80")

Overall Width (approx): 100 cm (39")

Weight: 1.5 kg (3.3 lbs)

MANGAR Rhino Lifting Cushion

Lifts a person in the supine position from the floor to the height of a bed or stretcher.

Maximum User Weight: 500 kg (1102 lbs/79 st)

Inflated Height (approx): 65 cm (26")

Overall Length (approx): 204 cm (80")

Overall Width (approx): 112 cm (44")

Weight: 7.5 kg (16.5 lbs)

The Mangar Patient Transfer System enables healthcare workers to lift and transfer patients with minimal moving and handling, while providing a stable, safe experience for the

The system consists of the Mangar Rhino Lifting Cushion, the Mangar Swift Transfer Slide, and the Mangar Airflo Duo Compressor.

- Mains and battery operated
- Lightweight anti-rip material
- Complete Supine Transfer System weighs under 20 kg



Excellent for infection control



Features

- 1. Roller Bag** - Fits the Rhino, Swift, Duo Compressor and Mains Power Supply with a total weight under 20 kg.
- 2. Infection Control** - Ready roller bag was designed during Covid-19 pandemic and offers a reliable, durable and clean way to manage the product between patient uses.
- 3. Lightweight, durable anti-rip material** - Easy to fold and return to bag after use.
- 4. Mains and battery powered** - Airflo Duo Compressor can be transported easily and used anywhere.
- 5. Patient Markings** - To show optimum position of patient on Rhino and Swift.

Product code: LSS403830 (Includes Swift & Rhino)



SUPINE
TRANSFER
SYSTEM

PROTECTING HEALTHCARE PROFESSIONALS

It is important to safeguard the health of not just the fallen person, but also the carer or healthcare worker. In the UK, HSE estimate 1.5 million days are lost to work related musculoskeletal injury annually, and the chance of an injury reduces by up to 41% when a lifting device is readily available. Injuries among healthcare workers can be costly not only to employers and employees and can also negatively impact the quality of care that a patient may receive. Consequences for the employee can include a loss of ability to perform duties, time off required and needing early retirement.

The Mangar ELK and Mangar Camel have been used for more than 15 years in a range of settings, such as care homes, housing associations, hospitals, telecare response teams, emergency services and in the community. The Mangar lifting cushions regularly support the work of healthcare professionals including care assistants, nurses, occupational therapists, physiotherapists and paramedics.

EASY TO USE:

No additional accessories are required to complete a lift - one size fits all. Very little staff training is needed.

EASY TO CLEAN:

Lifting cushions are made with a durable Riverseal® material with an anti-microbial coating. They are cleaned using a proprietary nonabrasive mild liquid cleaner.

EASY TO TRANSPORT:

Lifting cushions are battery powered, lightweight and portable.

CONTACT US

Help in person:

We regularly work with occupational therapists to assess clients needs. Our team of specialists will attend joint assessments and help select the lifting cushion that suits both the person who falls and their care giver.

*Please note, free in-home demonstrations are not available for all our products.



“Using the Mangar lifting cushions means our residents can be raised to their feet with minimal fuss. They are comfortable, dignified and don’t feel as though they are causing a scene and upsetting fellow residents. This means anxiety levels are kept to a minimum, which is vital for anyone with dementia.”

Delroy Bonnitto, Perry Tree Centre

CONTACT US TODAY

For more information visit



1300 133 120 | aidacare.com.au
sales@aidacare.com.au

We look forward to hearing from you!

Plus, we have online training courses available at aidacare.com.au





Mangar Lifting Cushions:

**Rhino & Swift Transfer Slide
Eagle / ELK / Camel**

Exclusively distributed by



1300 133 120 | aidacare.com.au



ABR139 290524

